

## Quick Start for IntegradWeb Internet Browser

Open Internet Explorer to your Integrad® Website.

Always answer “Yes” to security questions.

Type your username and password.

Click Login (or use Enter key).

**\*Note: All examinations should be viewed using Lossless Wavelet Compression**



Select an appropriate Tab, or use the binocular icon to search for patient exams.

Single click on the highlighted line to open a patient’s exam.

**To select a different layout**, click Layout on the main menu.

- Click “Load”, select from the list, click “Apply”.
- Note: If no layouts appear in Private list, try “Common”, or check “All Screen Sizes”.

**To page through images**, click on a series to activate it, then:

- Left/middle click.
- Scroll using mouse wheel.
- Drag left mouse button.
- Use up/down arrow keys or page up/page down keys.

**To adjust Window/Level:**

- Right click and drag to manually adjust W/L.
- Right click for pop-up menu, go to “Preset Windows”, select desired W/L.

***For additional options, right click for the pop-up menu.***

If Film View is active



Use the play buttons to page through images:



- Click Page Format on the Main Menu to select a different matrix display.

**For X-rays**, use the Magnifying Glass tool to view image at 1.0 zoom (100% resolution).

- Click icon to toggle between off and on.
- Click and drag the corner of the ROI to resize the viewing area.

**To close the study**, click on the done button



Or click on the down arrow to select from the menu.

Click the **Logout** button



to end session (frees the license for another user to login).